

PedCath Registered Labs

PedCath tracks the labs where the hemodynamics import utility receives files from. In order to import data files from a lab it is necessary to register it in PedCath by specifying the lab name.

PedCath supports lab registration from the following systems: -

- GE MacLab and CardioLab
- Siemens Sensis
- Philips Xper
- Merge Hemo

It is not necessary to register other lab types.

Follow these instructions (once from any workstation) to register labs with PedCath:

- 1) Open the Direct Data Import Screen
 - From the main menu choose: Tools > Import Cath Data.
- 2) Ensure at least one file from each lab you'd like to register is shown in the list. (Additional labs can be registered at a later time as well).
- 3) ***For PedCath 8.4.5 and below only:*** Click the Browse button at the top of the import screen next to the import path, and in the "Show files of type" drop-down choose a specific import type (rather than "All Licensed Import Files") and click OK.
- 4) The Register Labs screen may pop up automatically. If not click the **Lab Licenses** button.
- 5) Check off the labs you would like to import from in the "Additional Labs Found" box, up to the number of licenses you have available, and click **Register Labs**.



If you do not see the lab name shown on the registered labs or within the import grid (but the patient file is shown), the name may not be set in your hemodynamic system. If this is the case, check with your hemodynamic vendor for assistance setting that field. Any string that uniquely defines the lab within your institution would work (e.g. "MacLab Rm104").

In PedCath 8.4.6 and above, the lab name is displayed on the import listing to show where each file originated from.

To acquire additional lab licenses or for assistance, please contact Scientific Software at 434.293.7661 or support@pedcath.com.